


Elevation of the upper body:

improved elevation of the upper body: The stable ramp and the pillow ensure the necessary back support. The arm can be supported on the side with an arm rest. Comfortable support of the feet with a universal pillow. The roll serves to relax the knees.


- A #12. **FILL MED** universal ramp, 50 x 60 x 25 cm
- B # 1. **FILL PREMIUM** head pillow, 80 x 80 cm
- C # 1. **FILL PREMIUM** universal pillow, 40 x 80 cm
- D #12. **FILL MED** arm rest, 80 x 25 x 25 cm
- E # 8. roll, 40 x 15 cm

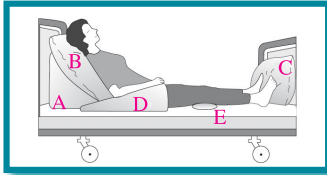

LATERAL BEDDING:

For better ventilation of the lungs if elevation of the upper body is not possible. The patient is supported with a back pillow to prevent slipping back. He lies on the 20 cm wide pane and keeps the pillow in place through his own weight. A small pillow is placed beneath the head to prevent twisting of the lumbar vertebrae. The arms can be supported on the side with a small pillow.


- A #2. **FILL MED** back support pillow, 60 x 80 cm with pane,
- B #1. **FILL MED PREMIUM** head pillow, 40 x 80 cm
- C #1. **FILL MED** universal pillow, 40 x 80 cm
- D #1. **FILL MED** bedding pillow, 35 x 40 cm

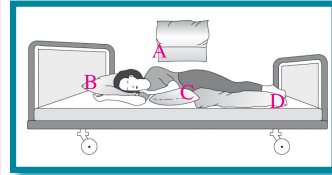

SITTING UP IN BED:

A seated position in bed can be realised by e.g. combining tiered bed and head pillows.


- A #7. **FILL MED** tiered bed pillow, 50 x 50 x 40 cm
- B #1. **FILL MED** universal pillow, 80 x 80 cm

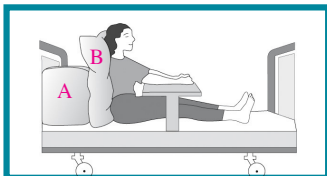

PILLOW BED:

Consisting of six FILL PREMIUM pillows. Vulnerable parts of the body like shoulder blades, spine, coccyx region and heels are „suspended“ where the pillow edges meet (lower regions).


- A-F #1. **FILL PREMIUM** universal pillow, with denser filling for soft bedding, 40 x 80 cm

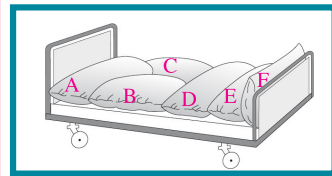

30° - OBLIQUE POSITION:

Low-risk position, no strain on sacrum and trochanter. Patients should be placed on a soft mattress (poss. MOLTOSLOW visco lining). Support head well with a pillow. Place patient in 30° oblique position by placing soft pillows beneath the body.


- A #1. **FILL PREMIUM** pillow, 40 x 80 cm
- B #1. **FILL MED** universal pillow, 40 x 80 cm
- C #1. **FILL MED** universal pillow, 40 x 80 cm
- D #2. **FILL MED** back support pillow, 40 x 80 cm with pane

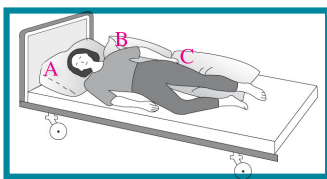

135° - OBLIQUE POSITION:

is often used as an alternative to prone position. Patients are placed on a soft mattress (poss. MOLTOSLOW visco lining). A large pillow is positioned below the torso to support the pelvis. A smaller pillow supports the head. The leg on top is supported with a universal pillow.


- A #1. **FILL PREMIUM** pillow, 80 x 80 cm
- B #1. **FILL PREMIUM** pillow, 40 x 80 cm
- C #1. **FILL MED** universal pillow, 40 x 80 cm

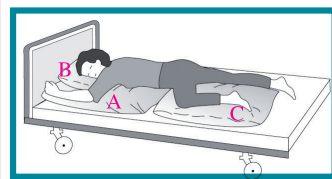

BEDDING WITH ROLL:

The **FILL MED** support roll is used for the most varied bedding purposes. In the field of basal stimulating care, the limits of space and body are conveyed via tactile stimulation.

- A #8. **FILL MED** Lagerungsrolle, Ø 20 cm, length 220 cm

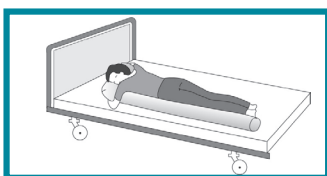

AVT-POSITION

The AVT position serves to suspend certain parts of the back after back surgery. The pillows can also be used to expand certain lung sections. As shown in the illustration, the support pillows are placed beneath the patient. Head and universal pillows are used in the region of the head and feet.

- A #1. **FILL PREMIUM** pillow, 40 x 80 cm
- B #1. **FILL MED** ATV-pillow, 20 x 80 cm
- C #1. **FILL MED** universal pillow, 40 x 80 cm

