



DGKS Dagmar Berger

Admission Diagnosis: 14 February Patient Data:

MS patient, in stage 9.5, 38-year-old completely immobile, incontinent, in poor general health. Home care provided by Sozialsprenkel.

Diagnoses:

Cachexia, has a PEG feeding tube, MS stage 9.5, Chronic bronchitis, anaemia, stage 4 to 5 decubitus With loculi at risk for sepsis, chronic UTI; The patient is also severely malnourished and has numerous deficiency syndromes: vitamin, mineral and protein deficits; he will be transitioned to hyper-alimentation.

Report- DGKS D. Berger (Nurse)

Due to severe malnourishment and almost 14 days of diarrhoea, the patient developed decubiti in the following areas with formation of loculi: on the sacrum at a depth of 6cm, stage 4-5; both trochanters at stage 4-5 and a depth of 6-8cm. This patient has another problem which is his allergy to the smallest quantity of latex. He cannot use an alternating pressure mattress because it contains latex. Thus he has a thermoelastic MOLTOSLOW liner, which allows for optimum distribution of pressure using his own body heat.

Though the patient received treatment with Fibrolan® ointment, hydrocolloid and silver alginate dressings etc. from February until 20 June of this year, his condition continued to deteriorate. That is when I started treatment with MOMOSAN-white.

Treatment Process:

Start of treatment 21 June

- 1st-3rd Day: change dressing every 6-8 hrs. (complete wound exudate absorption by MOMOSAN white STERILE)
- 4th-8th Day: change dressing every 12-18 hrs.
- After day 8: change dressing every 36-48 hrs.

This was the treatment protocol until the wound healed. The wound was cleansed with potentilla anserina and the wound edges were treated with zinc ointment. Photos were taken every 2 days. After just 14 days, the trochanteric and sacral loculi started to close. After 4 more weeks, the sacral loculi had closed completely.

Here, the skin thus has started to granulate (healing process). Today, the trochanters have a depth of 3-5mm and the depth of the loculi has decreased by a finger tip. Here treatment was continued with the same process for about 4-6 weeks because these decubiti were significantly more extensive.

Viewing the photos, it is clear to see how effective and easy it is to use MOMOSAN.



21.06.



04.07.



18.08.

Summary:

With a challenge like this, MOMOSANwhite has really shown its effectiveness. Easy to use, MOMOSAN white STERILE almost completely healed the wound in just 2 months without the need for any other product. Practical and visibly effective, MOMOSAN white STERILE provides a cost-effective alternative to other treatment methods.

Cost for Dressing Supplies:

3 pkg. MOMOSAN-white 14 sterile bandages
Art.Nr. 61905 € 95,90
6 pieces MOMOSAN-white pads 59x49x1 cm
from Art.Nr. 61121 € 2,82

Total Cost-For Use:

For 2 month

€ 301,92

MOMOSAN®

Practical Experience