


EVAC TWIN is a mattress cover for premium medical mattresses, which can also be used to evacuate patients using the stairs in the case of an emergency. All mattress cores from our range of medical and therapeutic mattresses can be incorporated in the evacuation mattress cover, which are intended for bed-ridden patients (as of an average/high expected risk of decubitus; as of existing stage 1 decubitus, with the exception of heavy duty systems). This two-in-one system has the advantage that appropriate attention can be given to the decubitus prophylaxis and therapy with the option of evacuation

These are the advantages:

- The rescue system for immobile patients/residents, can be used independent of the elevator
- Fast evacuation by **one** rescuer (also via the staircase)
- Secure positioning and fixation of the patient/resident using double belt system
- Ideal patient care during evacuation
- Easy and safe handling, suitable for patient weight up to a max. of 150kg
- Evacuation system **EVAC TWIN** is flame resistant
- **Same initial costs as an individual mattress + evacuation mat**



EVAC TWIN
two-in one at the same price
Mattress and evacuation mat
in one system

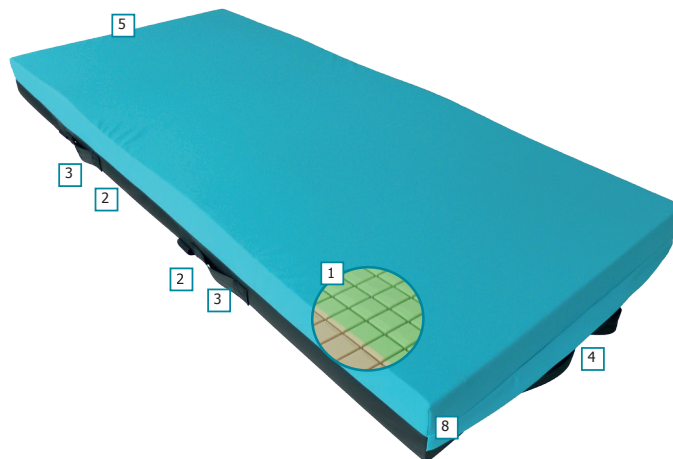

1		<p>Core: All mattress cores from our range of medical and therapeutic mattresses can be incorporated in the evacuation mattress cover, which are intended for bed-ridden patients (as of an average/high expected risk of decubitus; as of existing stage 1 decubitus, with the exception of heavy duty systems).</p>
2		<p>2 belt straps for the perfect fixation of the patient on the mat. The bedding (blanket, pillow) must remain in the bed and serves the secure and stable fixation of the patient and at the same time, protects the patient against hypothermia (see illustration 7). When not in use, these are stored in the strap supports on the bottom of the mattress (illustration 6).</p>
3		<p>4 carrying straps are intended to lift the patient out of the bed. If there is only 1 person available, the mattress can be pulled out with the patient using strap 4 (at the foot of the mattress).</p>
4		<p>Using the shorter strap at the foot of the mattress, the mattress can be pulled out of the bed by 1 person. In the further course of the evacuation, this strap is used to pull.</p>
5		<p>The longer strap at the head of the mattress is used to steer resp. brake, when 2 persons are handling the evacuation. This strap is folded and attached at the head of the mattress.</p>
6		<p>Bottom: Full surface, tear-resistant PUPOTEX™ 300 with strap support system for fastening straps while not in use.</p>
8		<p>Top cover: Made of time-tested PUNL® material (PU coated, bi-elastic, breathable fabric; impermeable to liquids, can be washed off and disinfected). The PUNL® top has a separable zipper and if necessary, can be easily individually replaced.</p>